









Cardiochek Running a Test


<p>1. Insert Memo Chip with a lot number that matches test strip vial lot number. Press either button to turn the Cardiochek ON. Analyser will display lot code. The WDHS team leader will demonstrate and assist with this step.</p>	
<p>2. When INSERT STRIP is displayed, hold by the raised lines and insert strip into the analyser as far as it will go</p>	
<p>3. When APPLY SAMPLE is displayed, apply whole blood sample with a capillary pipet to blood application window</p>	
<p>4. Within two minutes the result will appear on the display. Remove and discard strip. Do not add more blood to a test strip that has been used.</p>	
<div style="display: flex; align-items: center;">  </div>	

Instructions for Collecting and Dispensing Blood with a Capillary Blood Collector Pipet

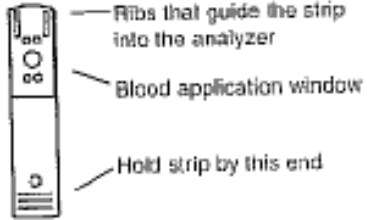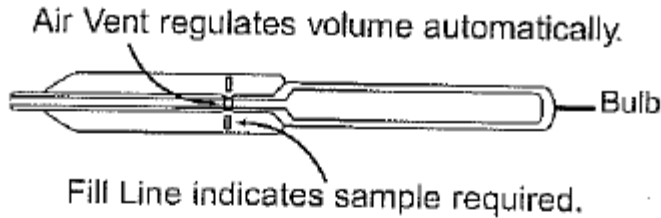

Filling


CAUTION! Capillary fills automatically. Hold by bulb, but never squeeze the bulb while collecting blood.*

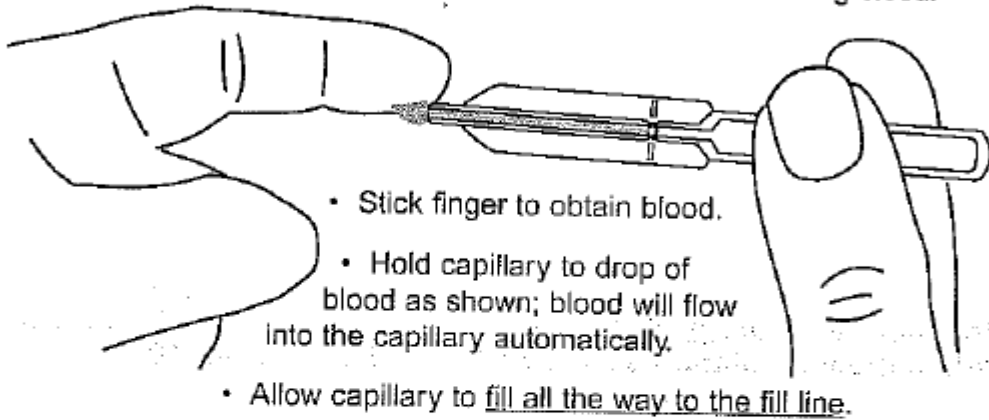

* Important: If you squeeze the bulb when collecting the blood, discard and use a new capillary to collect another sample.

Dispensing

- Insert test strip into CardioChek® ribbed side up.
- Hold capillary by the bulb end. Position above the blood application window on the test strip.
- Place a finger over the air vent and squeeze the capillary bulb firmly to dispense blood onto the test strip.

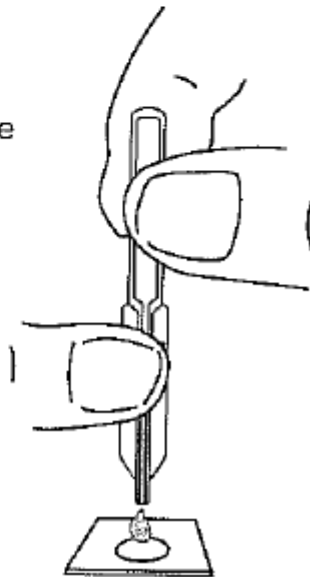

CAUTION! Never touch the test strip with the tip of the capillary